




Ameda Milk Collection Set User Instructions

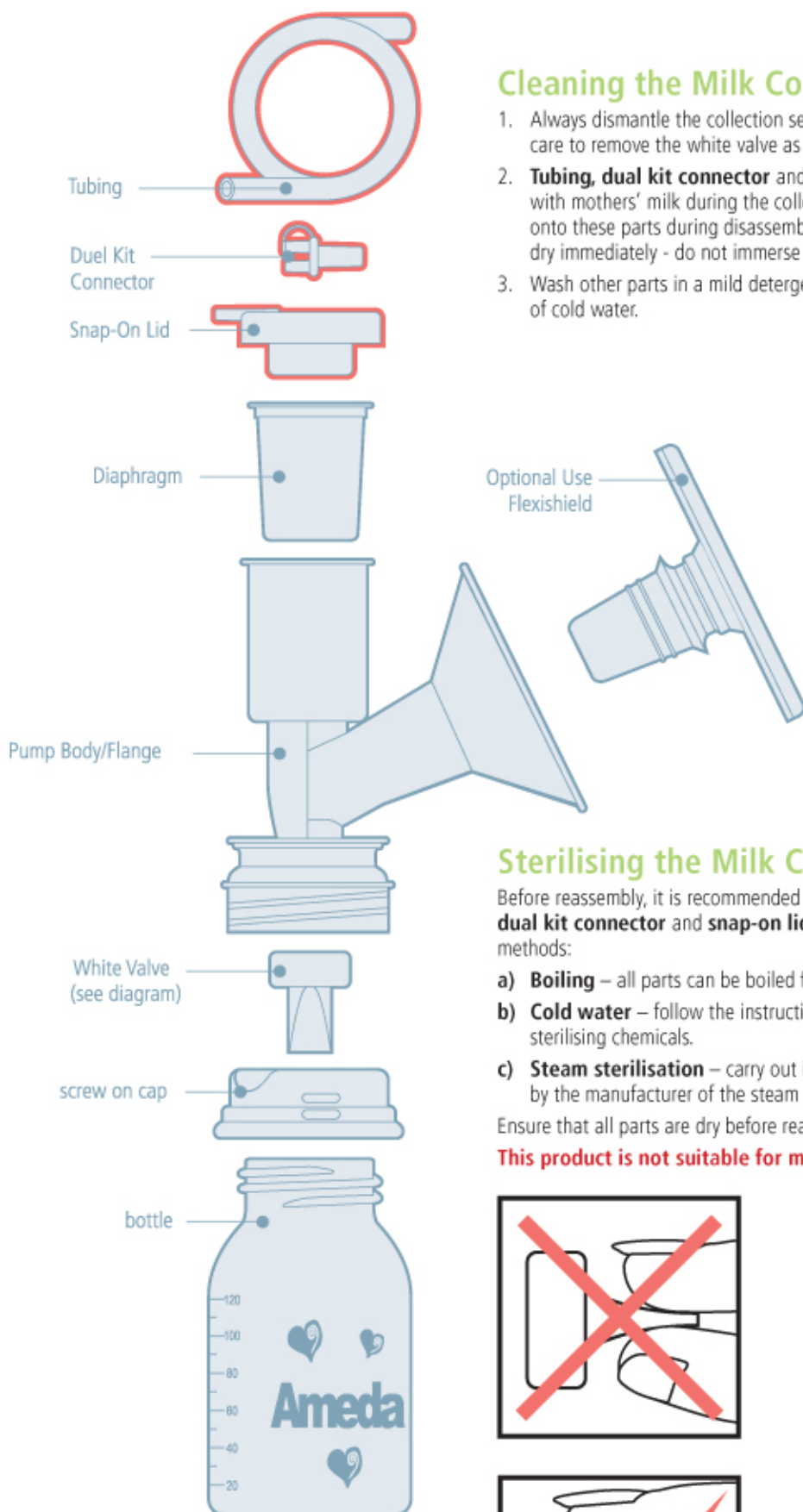

Cleaning the Milk Collection Set

1. Always dismantle the collection set prior to cleaning and sterilising taking care to remove the white valve as shown in diagram
2. **Tubing, dual kit connector** and **snap-on lid** do not come into contact with mothers' milk during the collection cycle - in the event of spillage onto these parts during disassembly, wash with mild detergent, rinse and dry immediately - do not immerse these parts in water.
3. Wash other parts in a mild detergent and rinse clean with a stream of cold water.


Sterilising the Milk Collection Set

Before reassembly, it is recommended that the set, except for the **tubing, dual kit connector** and **snap-on lid**, is sterilised using one of the following methods:

- a) **Boiling** – all parts can be boiled for 20 minutes.
- b) **Cold water** – follow the instructions stated by the manufacturer of the sterilising chemicals.
- c) **Steam sterilisation** – carry out in accordance with the instructions set by the manufacturer of the steam steriliser.

Ensure that all parts are dry before reassembling the set.

This product is not suitable for microwave sterilisation

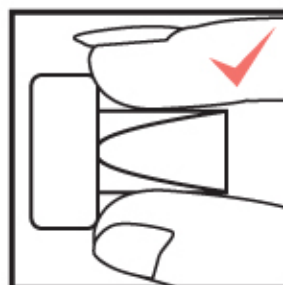

Diagram

"The products featured in this literature, and their packaging, do not contain natural rubber latex nor do their suppliers add natural rubber latex to their production processes".



Ardo medical Ltd.
Unit 1 Belvedere Trading Estate
Taunton, TA1 1BH
United Kingdom
T: +44 (0)1823 336362
F: +44 (0)1823 336364
www.ardomedical.co.uk
info@ardomedical.co.uk